




Turn your File into a Magic Wand

- Chemically cleans and removes smear layer
- Lubricates and reduces the risk of instruments fracturing

Ce le control

GLYDE™ FILE PREP

Makes root canal treatment easier whichever preparation method is used

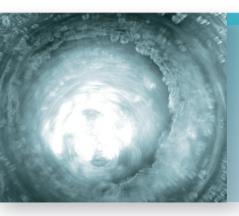

- Reduces the risk of instruments fracturing by acting as a lubricant
- · Helps to flush out debris
 - EDTA, a demineralising agent, softens dentine making calcified canals easier to shape.
 - The combination of urea peroxyde causes an effervescent reaction which helps to remove smear layer in the canal up towards the crown.




 Canal walls free of smear layer due to EDTA and its softening effect

 Using EDTA, a chelating (demineralising) agent, clean canal walls, completely free of smear layer are obtained and calcified canals are easier to shape.

.....smear layer



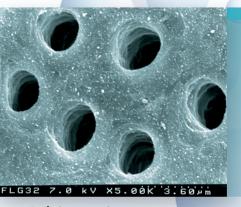

> Prof. Giuseppe Cantatore

- Chemically cleans and removes smear layer thanks to urea peroxyde
- Complete removal by rinsing thanks to its completely water-soluble formulation
 - Urea peroxyde, an oxidising agent also used in bleaching processes, chemically cleans and removes anaerobic bacteria in the canal flora.
 The formulation, which is completely water-soluble, gives the mixture its gel consistency.

Use of the GLYDE™ FILE PREP

Rinse with sodium hypochlorite and make the first entry into the canal using the first manual or rotary instrument.

Apply the Glyde™ File Prep straight into the cavity using the disposable tip or the single-use pack.

Syringe and interchangeable tips



Reinsert the first instrument and begin to prepare the canal.

4 a. No cross-contamination with **disposable doses.**

Rinse with hypochlorite and apply the Glyde™ again into the access cavity (also available in very practical disposable doses).

Continue preparing the canal alternating with sodium hypochlorite and Glyde™ between each instrument.

Once the canal has been shaped, rinse it with plenty of sodium hypochlorite until the effervescence subsides.

Dry and fill.

cal modifications on our products are not subject to notifications. Photos of our instruments are not contra o 02 24 E/02/1900 - undeped 07/2012

Turn your File into a Magic Wand

- Reduces the risk of instruments fracturing by acting as a lubricant
- Helps to flush out debris
- Canal walls free of smear layer due to EDTA and its softening effect
- Chemically cleans and removes smear layer thanks to urea peroxyde
- Complete removal by rinsing due to its completely water-soluble formulation.



