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5- **CONTRA-INDICATIONS**: Known allergy to iodine and its derivatives. Do not use if the aspect of the powder has changed (color, agglomerate, ...).
**6 - INSTRUCTIONS FOR USE**: Shake the jar well before use. Add 4 spoons of SEALITE REGULAR Powder to 7 drops of SEALITE Liquid, mixing with a spatula on a frosted glass plate for approximately 1 minute until a creamy, homogeneous paste is obtained with good pickup by the lentulo.
**7 - STORAGE**: Keep in the original container at a temperature between + 5°C and +25°C protected from moisture. After opening: close immediately and tightly SEALITE REGULAR Powder. Do not use past the expiry date indicated on the outer package. Shelf-life: 2 years from the manufacturing date.
**8 - DISPOSAL**: Recycle or dispose of according to current legislations, rather by a waste collector or an approved company. Do not dispose of in the environment. Professional dental product.
January 2018

## SEALITE LIQUID

**1 - PRESENTATION**: 1 bottle with 10ml of SEALITE Liquid. Bottle for multiple use.
**2 - INDICATION**: Liquid for Permanent sealing cement for root canals.
**3 - COMPOSITION**: Eugenol; Excipients.
**4 - PRECAUTIONS FOR USE**: Glass bottle: handle with care to avoid breakage. Wear suitable gloves and goggles when handling the product. WARNING. Skin Sens. 1. May cause an allergic skin reaction. Eye Irrit. 2. Causes serious eye irritation. Avoid breathing dust/fume/gas/mist/vapours/spray. Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: wash with plenty of water/soap. IF IN EYES: rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: get medical advice/attention. If skin irritation or rash occurs: get medical advice/attention. Dispose of contents/container to an authorized center. Contains eugenol. May produce an allergic reaction.
**5 - CONTRA-INDICATIONS**: Known allergy to eugenol. Do not use if the aspect of the liquid has changed (colour, odour, ...).
**6 - INSTRUCTIONS FOR USE**: Add 4 spoons of SEALITE REGULAR Powder to 7 drops of SEALITE Liquid, mixing with a spatula on a frosted glass plate for approximately 1 minute until a creamy, homogeneous paste is obtained with good pickup by the lentulo.
**7 - STORAGE**: Keep in the original container at a temperature between + 5°C and + 25°C protected from light. After opening: close immediately and tightly SEALITE Liquid after use. Do not use past the expiry date indicated on the outer package. Shelf-life: 2 years from the manufacturing date.
**8 - DISPOSAL**: Recycle or dispose of according to current legislations, rather by a waste collector or an approved company. Do not dispose of in the environment. Professional dental product.
January 2018

**CE**

0459

## SEALITE REGULAR KIT

**1 - PRESENTATION**: 1 jar with 25g of SEALITE REGULAR Powder, 1 bottle with 10ml of SEALITE Liquid, 1 spoon (powder), 1 dropper. Jar and bottle for multiple use.
**2 - INDICATION**: Powder and Liquid for Permanent sealing cement for root canals.
**3 - TECHNICAL CHARACTERISTICS**:

CRITERIA TESTED	SEALITE REGULAR Powder + SEALITE Liquid
Powder/Liquid ratio	4 spoons / 7 drops
Creep (diameter in mm)	25 to 35mm
Working time	≥ 30min
Setting time (37 <span> </span> °C - water bath)	15 to 50min
Film thickness (µm)	≤ 50
Radiopacity	≥ 3mm of aluminium
Solubility and disintegration	≤ 3%

4- **PRECAUTIONS FOR USE**: Check the length of the canal before filling so as to avoid forcing back the paste in the peri-apex. Prepare the paste immediately before use to avoid any contamination. Do not reuse a cement already prepared for another patient (risk of contamination). To avoid possible coloration of the dentin, do not fill the pulp chamber with the cement and apply an isolating cavity liner. In the absence of teratogenesis data, use during pregnancy is only possible in the absence of any other alternative of root canal treatment. Given the very local use and low quantity of the product, it is unlikely to be present in breast milk. However, in the absence of specific studies, the product should only be used during breastfeeding if the practitioner deems this necessary. Avoid using this product after pulpectomy of immature teeth. Do not swallow. Keep out of reach of children.

5- **CONTRA-INDICATIONS**: Do not use in children (immature teeth). Known allergy to iodine and its derivatives and/or to eugenol.

6- **INSTRUCTIONS FOR USE**: Shake the jar (SEALITE REGULAR Powder) well before use. Immediately before use, add 4 spoons of SEALITE REGULAR Powder to 7 drops of SEALITE Liquid, mixing with a spatula on a frosted glass plate for approximately 1 minute until a creamy, homogeneous paste is obtained in sufficient quantity for the root canal and with good pickup by the lentulo. Prepare the canal for filling according to standard techniques, then fill the canal. The cement produced must be used in combination with gutta-percha cones according to standard endodontic techniques. Take an X-ray to check that the canal is properly filled.
**7 - DISPOSAL**: Do not dispose of in the environment. Professional dental products.

## SEALITE REGULAR POWDER

**1 - PRESENTATION**: 1 jar with 25g of SEALITE REGULAR Powder. Jar for multiple use.
**2 - INDICATION**: Powder for Permanent sealing cement for root canals.
**3 - COMPOSITION**: Diiodothymol; Zinc oxide; Radio-opacifier; Excipients.
**4 - PRECAUTIONS FOR USE**: Glass jar: handle with care to avoid breakage. Wear suitable gloves and goggles when handling the product. WARNING. Skin Sens. 1. May cause an allergic skin reaction. Aquatic Acute 1. Very toxic to aquatic life. Aquatic Chronic 1. Very toxic to aquatic life with long lasting effects. Avoid breathing dust/fume/gas/mist/vapours/spray. If skin irritation or rash occurs: get medical advice/attention. Wear protective gloves/protective clothing/eye protection/face protection. Avoid release to the environment. Collect spillage. Dispose of contents/container to an authorized center. Contains diiodothymol and rosin. May produce an allergic reaction.

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