<div>ESPE</div> <div>Ketac™ Cem Easymix</div> <div>Glass Ionomer Luting Cement</div> <div>Glasionomer-Befestigungszement</div> <div>Ciment verre ionomère pour scellement</div> <div>Cemento vetroso ionomero per fissaggio</div> <div>Cimento de vidro ionómero para fixação</div> <div>Glasionomer-bevestigingscement</div> <div>Υαλοίονομερήs κοvία συγκολλησων</div> <div>Glasjonomer-fastsättningscement</div> <div>Lasi-ionomeeri-kiinnityssementti</div> <div>Glasionomer-retentionscement</div> <div>Glassionomersement til krone-bro sementering</div>	
	
<div> <div></div> <div>Instructions for Use</div> </div> <div> <div></div> <div>Gebrauchsinformation</div> </div> <div> <div></div> <div>Mode d'emploi</div> </div> <div> <div></div> <div>Informazioni per l'uso</div> </div> <div> <div></div> <div>Información de uso</div> </div> <div> <div></div> <div>Instruções de uso</div> </div>	<div> <div></div> <div>Gebruiksinfo</div> </div> <div> <div></div> <div>Οδηγίες χρήσης</div> </div> <div> <div></div> <div>Bruksanvisning</div> </div> <div> <div></div> <div>Käyttöinfo</div> </div> <div> <div></div> <div>Brugsanvisning</div> </div> <div> <div></div> <div>Bruksinformasjon</div> </div>
<div> <div></div> <div>Made in Germany</div> </div> <div> <div></div> <div>3M ESPE AG</div> </div> <div> <div></div> <div>Dental Products</div> </div> <div> <div></div> <div>D-82229 Seefeld - Germany</div> </div>	<div> <div></div> <div>CE</div> </div> <div> <div></div> <div>0123</div> </div>
<div> <div></div> <div>3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.</div> </div> <div> <div></div> <div>3M, ESPE, Alkaliner and Ketac are trademarks of 3M or 3M ESPE AG. © 3M 2004. All rights reserved.</div> </div>	<div> <div></div> <div>4400073946501</div> </div>

<div> <div></div> <div>Instructions for Use</div> </div> <div> <div></div> <div>Gebrauchsinformation</div> </div> <div> <div></div> <div>Mode d'emploi</div> </div> <div> <div></div> <div>Informazioni per l'uso</div> </div> <div> <div></div> <div>Información de uso</div> </div> <div> <div></div> <div>Instruções de uso</div> </div>	<div> <div></div> <div>Gebruiksinfo</div> </div> <div> <div></div> <div>Οδηγίες χρήσης</div> </div> <div> <div></div> <div>Bruksanvisning</div> </div> <div> <div></div> <div>Käyttöinfo</div> </div> <div> <div></div> <div>Brugsanvisning</div> </div> <div> <div></div> <div>Bruksinformasjon</div> </div>
---	---

<div> <div></div> <div>Made in Germany</div> </div> <div> <div></div> <div>3M ESPE AG</div> </div> <div> <div></div> <div>Dental Products</div> </div> <div> <div></div> <div>D-82229 Seefeld - Germany</div> </div>	<div> <div></div> <div>CE</div> </div> <div> <div></div> <div>0123</div> </div>
<div> <div></div> <div>3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.</div> </div> <div> <div></div> <div>3M, ESPE, Alkaliner and Ketac are trademarks of 3M or 3M ESPE AG. © 3M 2004. All rights reserved.</div> </div>	<div> <div></div> <div>4400073946501</div> </div>

<div> <div></div> <div>3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.</div> </div> <div> <div></div> <div>3M, ESPE, Alkaliner and Ketac are trademarks of 3M or 3M ESPE AG. © 3M 2004. All rights reserved.</div> </div>	<div> <div></div> <div>4400073946501</div> </div>
---	--

ENGLISH

Product Description

Ketac™ Cem Easymix, manufactured by 3M ESPE, is a fixing cement for mixing by hand.

For details on Alkaliner™, manufactured by 3M ESPE, please refer to corresponding instructions for use. The instructions for use on the product concerned must be kept available for the length of time it is employed.

Indication

- Cementation of inlays, onlays, crowns, and bridges made from metal or metal-ceramics or covered with composite veneer material
- Cementation of inlays, onlays, crowns, and bridges made from composite or ceramics provided these are suitable for conventional cementing
- Cementation of pins and screws provided these are suitable for conventional cementing
- Luting cement for use on orthodontic strips
- Relinings

Preparation

- To provide for optimal adhesion, carefully clean enamel, dentin, and metal surfaces.
- Blow the cavity dry in only 2–3 short intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry!
 - **The cavity should be just so dry that the surface has a matt shiny appearance.** Excessive drying with compressed air can negatively affect adhesion or result in post-operative sensitivity after seating the restoration.
- The metal surfaces should be dried well with compressed air.
- Avoid further contamination by saliva.

Pulp Protection

The glass ionomer cement must not be applied directly to dentin – e.g. on inlays – situated close to the pulp or to the exposed pulp. Prior to taking the impression, cover areas close to the pulp using a hard-setting calcium hydroxide preparation, e.g. Alkaliner.

Dosage

- Shake bottle to loosen powder.
- Screw off cap and replace by dispensing dropper.
- For cementing work, the standard powder-to-liquid mixing ratio is 3.8 : 1 w/w, this corresponding to 1 level spoonful of powder to two drops of liquid.
- Skim off spoon on plastic segment. Do not compress the powder.
- Dose powder and liquid next to each other on a mixing pad or glass slab.
 - Keep bottle upright while dispensing.
 - Crystallized liquid must not be allowed to form on the dispensing dropper.
- For relinings more powder may be added according to the desired consistency.
- Carefully seal bottles after use.

If the mix is too thin (due to insufficient dosing of powder) this may produce symptoms of pulpitis.

Mixing

Working of Ketac Cem Easymix should proceed at a room temperature of 20–25°C.

- Mix using a metal or plastic cement spatula.
- Add the powder to the liquid in one portion.
- On principle a sufficient quantity of powder should be added to the liquid to produce viscid consistency.
- The quantity of cement required for a crown should just about drip from the spatula.
- Continue to smooth out the paste until a homogeneous mixture is obtained.

Application

- During application, water and saliva must be prevented from entering the working area.
- Apply a thin coat of cement to the inside of the crown and stump.
- Afterwards proceed with restoration.
 - Avoid overfilling of the crown. Applying hydrostatic pressure buildup in closely adjoining crowns may cause pulp discomfort.

Times

The following time scale applies at a room temperature of 23°C and 50% relative atmospheric humidity:

Mixing	0:30
Application incl. mixing	3:10
Setting from begin of mix	7:00

Higher temperatures shorten the working time, whereas lower temperatures will extend this period (e.g. mixing on a cooled glass slab). A higher amount of powder also brings about a shorter working time. Exceeding the working time causes loss of adhesion to enamel and dentine.

Removal of Excess

- Remove excess using a Heidemann spatula and/or probe 6–8 min. after beginning of mix.

Incompatibilities

The product may cause allergic reactions to sensitive persons. If allergic reactions occur, use of the product should be discontinued and the product completely removed. Pulp complaints may be experienced in rare cases, in particular where working instructions are not properly observed.

Storage and Shelf Life

Keep the powder in a dry place away from damp. Carefully rseal the powder bottle after each use. Store the product at 15–25°C/59–77°F. Do not use after the expiry date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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